PROCTOWASH™ Rectal Washout Kit

Instructions for Use

The Proctowash is a closed irrigation and collection system for rectal washouts.

Components

Handpiece

Waste Tubing

Waste Container

Irrigation Set

How to Use

1. Screw Proctowash Handpiece onto one end of Waste Tubing (if not already attached), and screw Waste Container onto other end of Waste Tubing.
2. Lubricate blue tip of Handpiece and insert into anus of patient.
3. Insert formed tip of Irrigation Set through irrigation entry port in Handpiece, and advance through Handpiece and into rectum of patient.
4. Irrigate with solution of choice. Sufficient length of tubing is provided for advancing into distal colon if desired.
5. Effluent drains through holes in tip of Handpiece, and down Waste Tubing to Waste Container.
6. Withdraw Irrigation Set and discard.
7. Withdraw Handpiece from patient, and empty residual effluent from Waste Tubing into Waste Container.
8. Detach Waste Container from Waste Tubing, add solidifying solution if desired, seal with cap provided, and discard.

Contamination risk if reused.

Any serious incident that occurs in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

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INTERMARK MEDICAL INNOVATIONS LTD